

Safe `n` Sit®

English

Introductions for Use

Description

SafenSit has a flexible neck with an adhesive tip which allows the dentist to safely hold the restorations while transporting and placing in the patient's prepared teeth so it can easily and quickly sit on any small object.

Indication

SafenSits are used to pick up, hold, and try-in small objects of all kinds, such as inlays, onlays, crown, veneers, bracket, and implants made of e.g. gold metal or Empress.



Caution

- The adhesive tip of SafenSit has a very strong adhesive power so it is enough touching slightly on the surface of object. Putting pressure strongly on the surface of object is not recommended.
- Dental treatment involves the risk of small objects being aspirated. It is advisable to apply a rubber dam.

Storing

- Do not use SafenSits after the date of expiration.
 - Room temperature storage.
 - Protect from exposure to direct sunlight.
 - Shelf life; see date of expiration.
- Keep out of the reach of children.**

Manufacturer

UTIC SAFEnSIT
<http://www.safensit.com>
E-mail: utic2011@yahoo.com, safensit@gmail.com
Tel: +1 520 413 8063, +1 702 347 8813, +1 520 233 1312 (mobile)
+82 2 542 8884, +82 10 2847 8819 (mobile)
Fax: +82 2 542 8062



Safe `n` Sit®



UTIC SAFEnSIT
<http://www.safensit.com>
Designed by UTIC Corporation
AZ USA
Made in Korea
E-mail: safensit@gmail.com